



TOREASY
2871 avenue de l'Europe
69140 RILLIEUX LA PAPE
tel: 06 61 80 22 22 - fax: 09 72 40 43 55
www.toreasy.com - info@toreasy.com


How to Use TorEasy/ TorAxis for IOS

Step 1 : DATAS

< Retour	toreasy	Exporter
Nom	FOREST	
Prénom	Pierre	
Date de nai...	15/06/1980	
Sexe	Masculin	
Pays	France	
Œil	Œil Gauche	
Implant	A4TW0T	
Angle	58	
Voir la photo		

Step 2 : Align patient's head

Make sure that your patient's head is placed horizontally

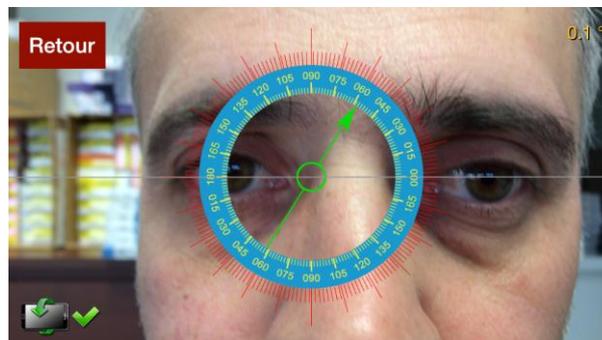




TOREASY
2871 avenue de l'Europe
69140 RILLIEUX LA PAPE
tel: 06 61 80 22 22 - fax: 09 72 40 43 55
www.toreasy.com - info@toreasy.com


Step 3 : Take your snapshot

Just touch the screen, and it's done!

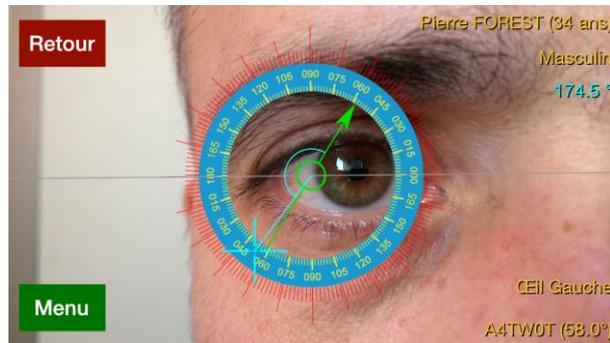


Step 4 : Enlarge and center your picture

Tag a blod vessel or an ink mark with the blue cross, read the value in blue (-76.2° in this case)

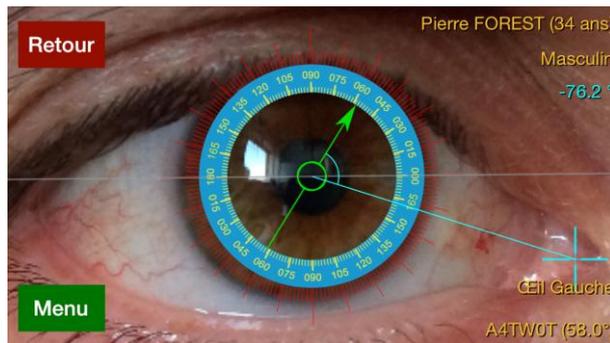




Step 5 : Use the menu to add contrast, save the cliché.

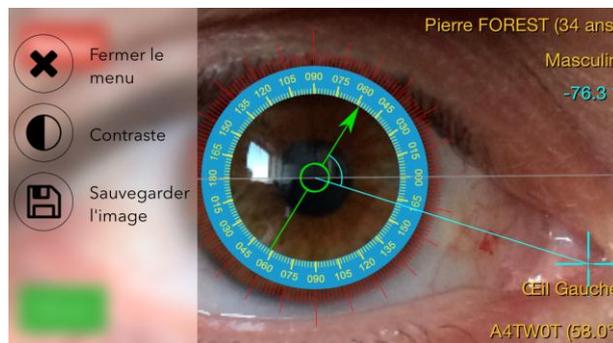



TOREASY
2871 avenue de l'Europe
69140 RILLIEUX LA PAPE
tel: 06 61 80 22 22 - fax: 09 72 40 43 55
www.toreasy.com - info@toreasy.com

Step 6 : use the menu to reverse the picture, and to use Torearing

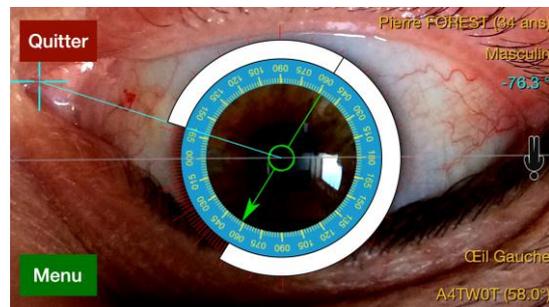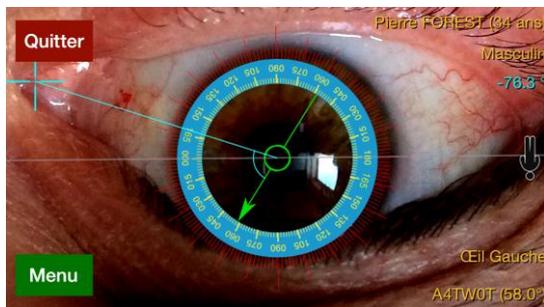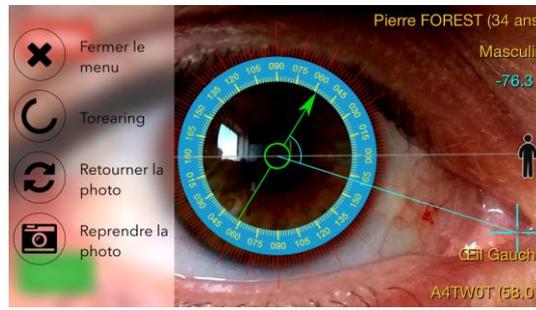

The green arrow, combined with mendez ring or Torearing will give you the IOL axis

Step 7 : Quit and Export your picture to your patient's file



Info and video: www.toreasy.com

PS : Toreasy can also be used pre op through the slitlamp to mark the cornea before the surgery, or to check iol position after the surgery.

Toreasy is available for Android, Iphone 4/4S/5/6, and Ipad